Operating Notes

Lumify Diagnostic Ultrasound System July 15, 2020

4535 620 24941 Rev A

About These Operating Notes

Operating notes contain information that clarifies certain system responses that might be misunderstood or cause user difficulty. Review the operating notes and keep a copy with the system manuals. If appropriate, post the operating notes near the ultrasound system.

Labeling Information

For your records, retain the *Quick Guide* included in shipment; it contains the labeling information for the Lumify Diagnostic Ultrasound System.

CE Mark and EC Rep Address

Philips Ultrasound, Inc., is transitioning from the CE0086 Notified Body (UK) CE Mark to a new CE2797 Notified Body (EU-27) CE Mark, effective March 1, 2019. Additionally, our European Union Authorized Representative (EC Rep) has a new street address:

Philips Medical Systems Nederland B.V. Veenpluis 6 5684 PC Best The Netherlands

During the transition, you may see the previous CE Mark and address on Instructions for Use and on the ultrasound system and its accessories.

Using the System

- Fast Flow and Slow Flow controls might overlap slightly in Color mode. This most often appears on 5th generation iPad mobile devices in landscape orientation. This does not affect the functionality of the controls.
- (iOS devices only) When you manually correct the patient Date of Birth field on the Patient
 Info form, clearing and reentering all four digits of the year results in an error. For best results,
 reenter the entire date.
- (iOS devices only) The navigation bar, including the Back control, is missing from the Contact
 Information form (accessed from Customer Information in Settings). To leave the page, you
 must enter your customer information, and then touch Continue.
- (iOS devices only) Rapidly touching **Save** multiple times when you scan might cause the Lumify software to shut down unexpectedly.
- (iOS devices only) The Lumify Power Module (LPM) might enter sleep mode if you exit the Lumify application or allow the mobile device to enter sleep mode. If you try to resume the



scan with the LPM in sleep mode, the **LPM Connection Error** message appears. To wake the LPM, press the button on the back of the LPM.

Review

- (iOS devices only) When you scroll to the bottom of a long list of saved exams, the last exam in the list might be cut off by the edge of the screen.
- (iOS devices only) When you export multiple exams to a local directory, you must confirm the
 export destination for each exam; you cannot use a single window to control the export of all
 of the exams.
- (iOS devices only) The sequence of selected images and loops that appears in an exam is different from the sequence that appears on Android devices.
- (iOS devices only) During exam export, the **Export Queue** shows the export status as **Aborted**. The export status should appear as **In Progress**.
- (iOS devices only) When you view a list of exams in the **Saved Exams** window and then touch **Select**, the list scrolls away from your intended selection.

User Information Changes

The following information replaces or adds to the information included on your *User Information* USB media.

Recycling, Reuse, and Disposal

The following information adds to information in the *User Manual*.

Perchlorate Material

In this system, perchlorate material is present in lithium coin cells or batteries. Special handling may apply to those items. For more information, see this website:

www.dtsc.ca.gov/hazardouswaste/perchlorate

Safety

The following information adds to or replaces information in the *User Manual*.

Symbols

Symbol	Standards and Reference	Reference Description	Additional Information
Ā	-	Product contains hazardous material. Dispose of properly. (Required by WEEE Directive; see EN 50419.)	
	ISO 7000-3725	Importer	

2 of 6 4535 620 24941

Symbol	Standards and Reference	Reference Description	Additional Information
	ISO 15223-1, Symbol 5.4.3	Consult instructions for use.	
Ĺ	ISO 7000-1641		
П	ISO 15223-1, Symbol 5.1.3	Date of Manufacture.	
\sim	ISO 7000-2497		
	ISO 15223-1, Symbol 5.1.1	Manufacturer.	
	ISO 7000-3082		
MS	IEC 60417	Country of Manufacture.	
[00]			
	ISO 15223-1, Symbol 5.1.4	Use-by date.	
><	ISO 7000-2607		
$\overline{\langle \lambda \rangle}$	IEC 60878, Symbol 1135	General symbol for recovery/	Do not throw away. Dispose of
P	ISO 7000-1135	recyclable.	in accordance with local, state, or federal laws.
CN	ISO 15223-1, Symbol 5.1.7	Serial Number.	
SN	ISO 7000-2498		
LOT	ISO 15223-1, Symbol 5.1.5	Batch code.	
LOT	ISO 7000-2492		
DEE	ISO 15223-1, Symbol 5.1.6	Catalog Number.	
REF	ISO 7000-2493		
STERILEEO	ISO 15223-1, Symbol 5.2.3	Sterilized using ethylene oxide.	
	ISO 7000-2501		
((European Commission	CE Mark of Conformity.	
CE	Medical Device Directive 93/ 42/EEC 2007/47/EC		
	EU MDR 2017/745, Article 20,		
	Annex 5		
CE	European Commission	CE0086 - CE Mark of	
0086	Medical Device Directive 93/	Conformity.	
	42/EEC 2007/47/EC		
	EU MDR 2017/745, Article 20, Annex 5		

4535 620 24941 3 of 6

Symbol	Standards and Reference	Reference Description	Additional Information
CG	European Commission	CE2797 - CE Mark of	
2797	Medical Device Directive 93/	Conformity.	
	42/EEC 2007/47/EC		
	EU MDR 2017/745, Article 20,		
	Annex 5		
(2)	ISO 15223-1, Symbol 5.4.2	Do not reuse.	
\bigcirc	ISO 7000-1051		
Α	ISO 15223-1, Symbol 5.4.4	Caution (ISO 7000-0434).	
∠į∖	ISO 7000-0434A		
	ISO 15223-1, Symbol 5.3.2	Keep away from sunlight.	
淡	ISO 7000-0624	,	
~ `	.53 / 555 552 7		
^	ISO 15223-1, Symbol 5.2.7	Non-sterile.	
NON STERILE	ISO 7000-2609		
\bigcirc	ISO 15223-1, Symbol 5.2.8	Do not use if package is	
(60)	ISO 7000-2606	damaged.	
	ISO 15223-1, Symbol 5.3.7	Temperature limit.	Indicates the temperature
	ISO 7000-0632		range (noncondensing) for
- / 			transport and storage. Does
	150453224.6.	Discontinue Desire	not apply to media.
(°/\	IEC 15223-1, Symbol 5.3.8	Humidity limitation.	The relative humidity range (noncondensing) for transport
کشک	ISO 7000-2620		and storage.
_			
~~	IEC 15223-1, Symbol 5.3.9	Atmospheric pressure	Atmospheric pressure range
(⇒•⇔)	ISO 7000-2621	limitation.	for transport and storage.
<u>uli</u> i	ISO 15223-1, Symbol 5.3.4	Keep dry.	
T	ISO 7000-0626		
_	ISO 15223-1, Symbol 5.3.1	Fragile, handle with care.	
Ī	ISO 7000-0621		

4 of 6 4535 620 24941

Symbol	Standards and Reference	Reference Description	Additional Information
	IEC 60878, Symbol 2794	Packaging unit.	
	ISO 7000-3724	Distributor	

Specifications

The following information adds to information in the *User Manual*.

Service Life

The Lumify system is designed to have a minimum of 3 years of service life. The service life of the Lumify system will vary depending on the usage and environmental conditions.

Philips Ultrasound, Inc.

22100 Bothell Everett Hwy, Bothell, WA 98021-8431 USA www.philips.com/ultrasound



© 2020 Koninklijke Philips N.V. All rights are reserved. Published in the USA. Reproduction or transmission in whole or in part, in any form or by any means, electronic, mechanical, or otherwise, is prohibited without the prior written consent of the copyright owner.

4535 620 24941 5 of 6

6 of 6 4535 620 24941