


PHILIPS

RESPIRONICS

DreamStation

PAP systems brochure

Veterans affairs

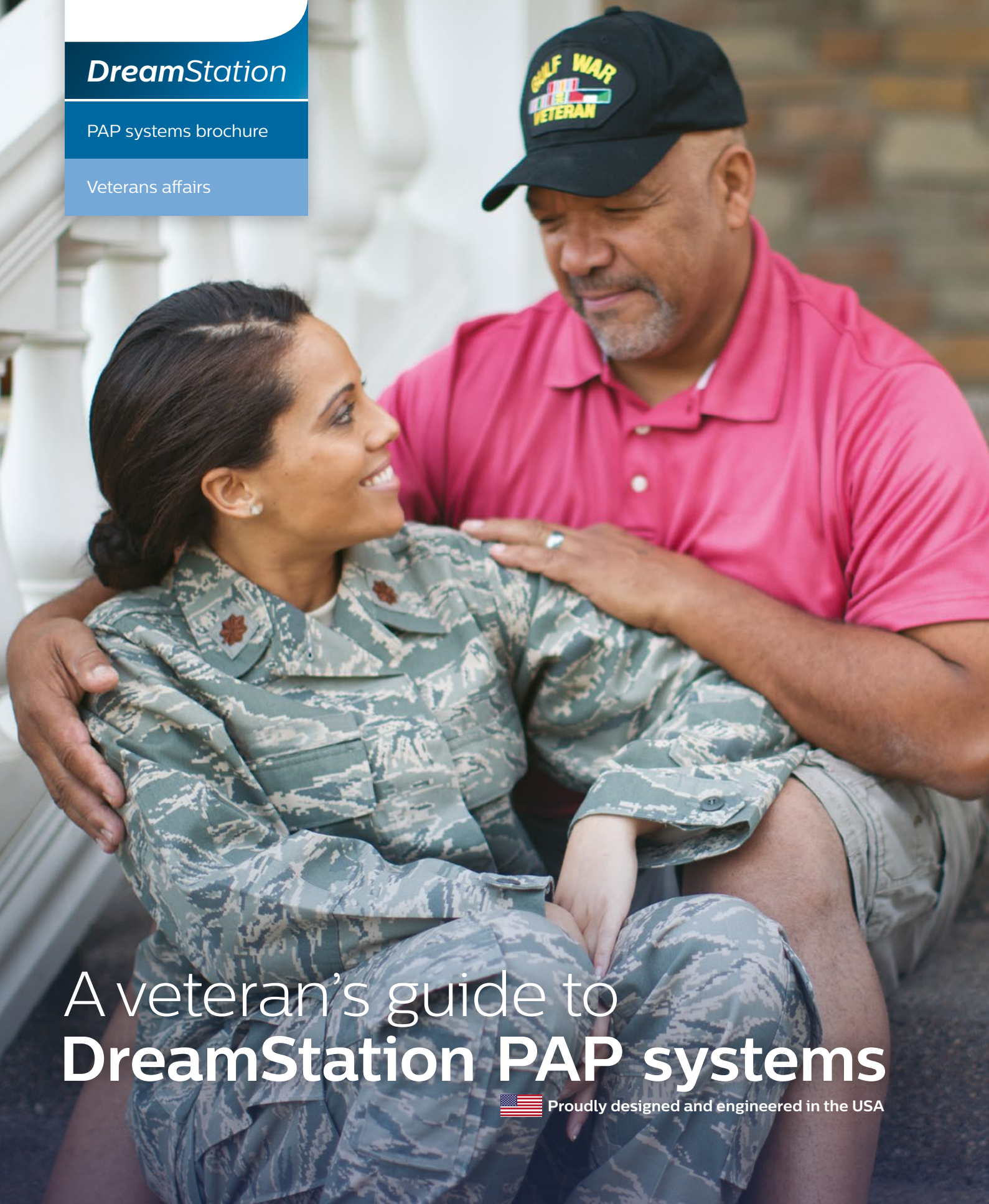

A veteran's guide to **DreamStation PAP systems**



Proudly designed and engineered in the USA

Natural. Comfortable. Sleep.

Philips Respironics DreamStation supports long-term patient use while helping to creating new efficiencies for your facility

Powerful patient-driven design

DreamStation was designed with feedback from hundreds of patients to provide a simple CPAP experience to help both new and experienced patients use therapy long-term. A suite of intuitive tools can be customized by healthcare professionals to choose the solution that's right for each patient.

Acclimating patients in the early days of therapy

EZ-Start can help patients gradually adapt to their prescribed level of therapy using an innovative pressure acclimation algorithm with automatic, personalized adjustments to CPAP pressure during the first 30 days.

SmartRamp helps users to fall asleep to lower pressures than those of standard ramp mode. The air pressure remains low until the patient experiences an obstructive respiratory event. Then SmartRamp responds with an auto algorithm to resolve the patient's apneas.

Make progress every day

Each morning, patients are greeted with Daily Progress Feedback — a summary of how far they've come — to motivate them to stick with their therapy. The intuitive, color display shows a simple trend of their nightly hours of use, followed by a summary of their last 30 days of good nights — those with more than four hours of use.

Sleep therapy success* at your patients' fingertips

Bluetooth® is included in every DreamStation device for easy connectivity to the DreamMapper patient self-management tool. DreamMapper helps patients take an active role in their sleep management by providing daily therapy progress, helpful troubleshooting videos and goal-setting tools. Patients can stay engaged and get the information they need fast, without having to use your staff's time to resolve common issues.

An Encore performance

DreamStation connects to our powerful **EncoreAnywhere** patient compliance management software. **EncoreAnywhere** makes it easy and efficient for sleep professionals to manage patient compliance and therapy.

Focus your time where it belongs

EncoreAnywhere features Adherence Profiler and Care Orchestrator. It's an optional, intuitive tool that helps you identify patients according to their probability of achieving adherence, helping focus your staff's time on the patients who have the greatest likelihood of success.

Proud to serve veterans

*In a retrospective review conducted by Philips Respironics of approximately 15,000 System One patients, patients who used SleepMapper, which has been rebranded to DreamMapper, demonstrated 22% greater adherence to the therapy than patients who did not use SleepMapper. To see a list of compatible DreamMapper devices, go to www.dreammapper.com/compatible.





Creating efficiencies that make a difference

Simplify device evaluation




Performance Check simplifies in-home device evaluation for easy remote troubleshooting. Designed to reduce the time and frustration involved in device troubleshooting, this robust tool allows you to guide the patient through an easy remote diagnosis, resulting in a simple "Pass/Fail" result to determine if the device needs to be returned for service.

Performance Check can also generate a report in our **EncoreAnywhere** patient management software tool, allowing you to troubleshoot patient concerns.

According to our research, 25% of returned devices are found to be operating normally.* Performance Check is designed to help reduce the number of normally functioning devices that are returned for service, saving you time and delivery costs.

Reduce manual adjustments

DreamStation can automatically adjust to changing patient needs, helping improve workflow processes and letting you focus your attention where it needs to be.

-  **CPAP-Check**
Checks on the user every 30 hours to determine if CPAP pressure is optimal. If not, it automatically adjusts to meet changing patient needs.
-  **Auto-Trial**
Delivers breath-by-breath auto-CPAP therapy for up to a total of 30 days per patient. Transitions to CPAP or CPAP-Check mode at end of Auto-Trial.
-  **Opti-Start**
Provides optimal starting pressure that can enhance patient comfort and reduce the likelihood of residual events at the beginning of therapy.

More connectivity options

to provide easy access to patient data



Only DreamStation offers Wi-Fi, Bluetooth® and cellular connectivity options. By providing the most comprehensive suite of connectivity choices among leading CPAP brands*, you can choose the options that are best for your sleep program. DreamStation's innovative design has the capability to adapt as technologies continue to advance.


Making it easy to reach your patients

- Wi-Fi delivers daily data to clinicians and provides coverage even in weak cellular areas
- Bluetooth is included in every device for easy connectivity with DreamMapper and Alice NightOne
- Cellular provides data on a daily basis and one hour after the patient has turned the device off

*Internal assessment of 2015 competitive CPAP data comparing to ResMed Airsense10/Aircurve10 platform and Fisher & Paykel Icon series platform.

The Philips Respironics DreamMapper helps patients take an **active role** in their therapy

With the DreamMapper self-management tool, patients can use the mobile and web-based features to engage with their therapy.

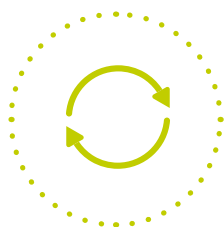

Sync devices

Data is easily synced and available through smartphones, tablets or computer



Track progress

Patients can monitor the progress of custom goals they set for themselves




View data

Patients can see their therapy data quickly and easily



Set reminders

Patients can set reminders for important tasks, including mask, tube and filter cleaning

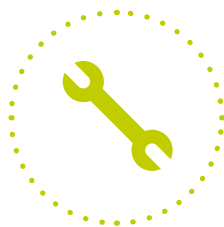

Troubleshoot

Troubleshoot common problems



Inspire patients

Tools to help encourage adherence

Performance and appeal

DreamStation offers a sleek, low-profile design. Its vibrant color screen displays daily progress and features easy-to-use menus.

DreamStation's top-mounted humidifier hose swivel and front-facing display make it easy for patients to operate while lying down or sitting up in bed. And, it's small and light, making it easy to pack for travel.

**30%
smaller**

The DreamStation PAP is 30% smaller than the Aironse10 PAP.*

**39%
lighter**

The DreamStation PAP is 39% lighter than the Aironse10 PAP.*

**63%
quieter**

The DreamStation PAP is 63% quieter than ResMed Aironse10.**

Hear it from **DreamStation users**

“Good-looking piece of equipment! It's modern looking, doesn't look like a medical machine. It's quieter, too.”

“I liked it as soon as I opened it! I liked the way it was laid out and I liked the size of it — it was small, easy to fit on a table along with a lamp and my phone.”

*Philips internal comparison, 2015, data on file

**Based on the average change in sound level during simulated patient breathing, across a range of pressure, expiratory relief, and humidifier conditions

Philips Respironics commitment to quality

Five-year DreamStation VA warranty

Philips Respironics is proud to offer a five-year warranty on all of our DreamStation sleep therapy products when purchased through First Nation Group, FSS, ECAT, MSPV and other authorized procurement channels.



Insight-driven innovation

Intuitive menu system

Graphics-based menu inspired by smartphone navigation for ease of use

Ambient light sensor

Automatically adjusts screen brightness for daytime or nighttime viewing




Integrated Bluetooth®

Wirelessly connects with DreamMapper or Alice NightOne portable sleep diagnostic system

Daily progress feedback

Displays previous three nights' therapy results to track progress

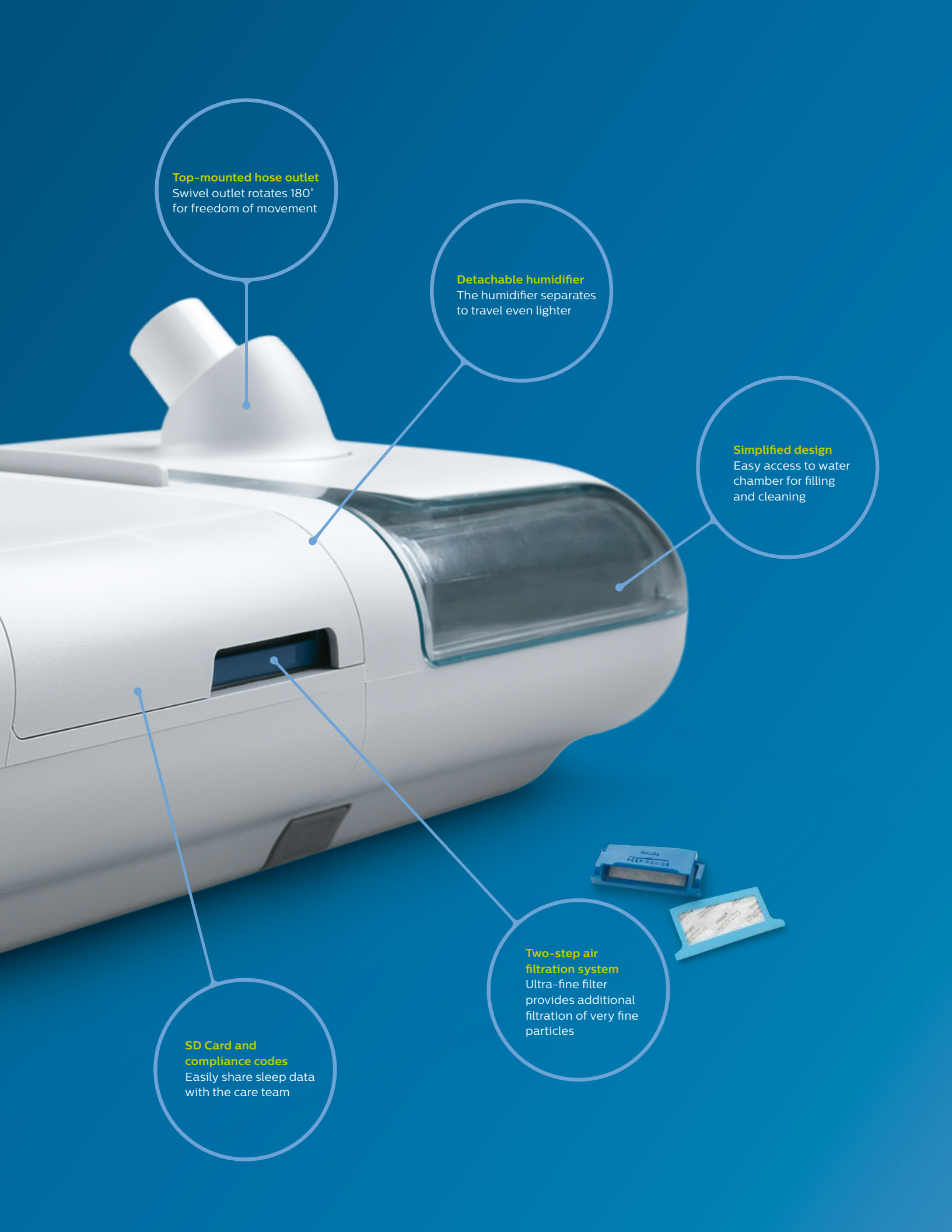

Top-mounted hose outlet
Swivel outlet rotates 180°
for freedom of movement

Detachable humidifier
The humidifier separates
to travel even lighter

Simplified design
Easy access to water
chamber for filling
and cleaning

**SD card and
compliance codes**
Easily share sleep data
with the care team

**Two-step air
filtration system**
Ultra-fine filter
provides additional
filtration of very
fine particles



Proudly designed and
engineered in the USA

Ordering information

DreamStation Auto CPAP Part number

DreamStation Auto CPAP	DSX500S11
DreamStation Auto CPAP with humidifier and cellular modem	DSX500H11C
DreamStation Auto CPAP with humidifier, heated tube and cellular modem	DSX500T11C

DreamStation Auto BiPAP Part number

DreamStation Auto BiPAP	DSX700S11
DreamStation Auto BiPAP with humidifier and cellular modem	DSX700H11C
DreamStation Auto BiPAP with humidifier, heated tube and cellular modem	DSX700T11C

DreamStation BiPAP autoSV Part number

DreamStation BiPAP autoSV	DSX900S11
DreamStation BiPAP autoSV with humidifier and cellular modem	DSX900H11C
DreamStation BiPAP autoSV with humidifier, heated tube and cellular modem	DSX900T11C

DreamStation BiPAP S/T Part number

DreamStation BiPAP S/T	DSX1030S11
DreamStation BiPAP S/T with humidifier and cellular modem	DSX1030H11C
DreamStation BiPAP S/T with humidifier, heated tube and cellular modem	DSX1030T11C

DreamStation BiPAP AVAPS Part number

DreamStation BiPAP AVAPS	DSX1130S11
DreamStation BiPAP AVAPS with humidifier and cellular modem	DSX1130H11C
DreamStation BiPAP AVAPS with humidifier, heated tube and cellular modem	DSX1130T11C

Humidification Part number

DreamStation humidifier	DSXH
DreamStation water chamber	1122520
DreamStation humidifier dry box inlet seal	1120613

Power management Part number

DreamStation 80-watt power supply	1118499
6 ft power cord	1005894
PAP Lithium Ion battery kit**	1124009
Li Ion battery, PAP device cable	1121736
Li Ion battery, cable adapter (50 series adapter, 2/each)	1127799
Li Ion battery case	1126947

Tubes Part number

15mm 6 ft standard tube	PR15
15mm 6 ft heated tube	HT15

Filters Part number

Pollen filter, reusable (1 per pack)	1122446
Ultra-fine filter, disposable (1 per pack)	1122447
Ultra-fine filter, disposable (2 per pack)	1122518
Ultra-fine filter, disposable (6 per pack)	1122519

Accessories Part number

Link module	1120293
Nonin SpO ₂ , assembly	1121694

Modems Part number

Cellular modem, oximetry capable	100603C
Cellular modem, non-oximetry	100604C
Wi-Fi accessory, oximetry capable	100700W
Wi-Fi accessory, non-oximetry	100705W

Travel accessories Part number

Universal PAP/laptop travel briefcase	1114784
DreamStation replacement carrying case	1121162
Shielded 12V DC power cord system (includes shielded DC cord and battery adapter cable)	1120747
DreamStation travel case, 65W power supply* w/ USB port	1120135
DreamStation travel case, 65W power supply* w/ USB port, 5/case	1124301

Mask and tubing bedside organizer Part number

Bedside organizer	1122135
Bedside organizer, 10/case	1124224

*DreamStation, mask, and tubing not included. Used with flow generator only. Cannot be used with a humidifier.

**PAP Lithium Ion Battery Kit for use with the Philips Respironics DreamStation and System One CPAPs, BiPAPs, autoSV, BiPAP AVAPS, and BiPAP S/T sleep therapy devices. Includes a rechargeable lithium ion battery pack, a PAP device cable, and a battery case.



Philips Respironics is proud to have its worldwide headquarters in the USA

Going further for you

The Philips Respironics RIST facility, located 20 miles east of Pittsburgh, PA, is home to our sleep and home respiratory division. This 172,000-square-foot greenfield facility was awarded LEED Gold certification in 2009 and was chosen by Assembly Magazine as the Assembly Plant of the Year.* Our USA-based team takes pride in their work because they know there's always a way to make life better for all of our customers – including veterans like you.



**Proudly designed and
engineered in the USA**

To order product, contact First Nation:

Fax: 734 929 1490
Phone: 800 541 2071
Email: orders@firstnationgroup.com

To contact Philips Respironics, call 800 345 6443 and follow the prompt as noted below:

- Customer service: press 1
- Repair/return/warranty questions: press 2, then press 7
- Product support: press 4

*As awarded by Assembly Magazine – September 26, 2011.



© 2019 Koninklijke Philips N.V. All rights reserved.
Specifications are subject to change without notice.
Trademarks are the property of Koninklijke Philips N.V.
or their respective owners.

www.philips.com/respironics

Caution: U.S. federal law restricts these devices to
sale by or on the order of a physician.

RRDPGH SH 6/12/19 MCI 4109091 PN 1127066
1010 Murry Ridge Lane, Murrysville, PA 15668
800 345 6443 • 724 387 4000