

Connected Trilogy **Solutions**

Powerful patient care



Connected Trilogy with Care Orchestrator



Trilogy wirelessly connects with the Care Orchestrator remote patient monitoring platform, which allows you and your care team to monitor and assess a patient's ventilation data from virtually anywhere.

Trilogy utilizes our cloud-based Care Orchestrator software solution to remotely monitor patient information via Bluetooth connection. It turns raw clinical data into actionable patient insights and delivers it directly to care teams via smart phone, tablet or PC.*

Connected Trilogy enables care teams to monitor patients remotely and create health rules to proactively notify you of a changing condition. This allows for fast and informed clinical decisions, including early intervention, which can help avoid unnecessary visits and readmissions and lower the cost of care.

Stay Connected

Connected Trilogy provides remote viewing of key ventilation parameters, alarm data, device performance and patient usage that can help you:



Minimize unnecessary, costly, unscheduled home visits



Identify patients who need intervention



Enable care team coordination with timely* ventilation information

*Automatically sent up to three times per day, plus on-demand manual data push option



You make the rules

With Connected Trilogy and Care Orchestrator, you can customize rules and notifications to remotely monitor and provide proactive notification of a patient's changing condition. So you can make fast, informed clinical decisions.

What are rules and reminders?

Rules are a set of parameters that run against downloaded data to generate tasks for clinical users. They enable clinical users to provide the right personalized care for patients in need.

Types of rules for Connected Trilogy

Health rules

Usage rules

Reminders are general purpose alerts that trigger to create tasks for clinical users.

What rules and reminders are triggered for a patient?

- ✓ **Usage rules** are triggered when the device has not downloaded any therapy data after a customized set number of days. Usage rules can be assigned based on data download method – via SD card or the Connected Trilogy service.
- ✓ **Health rules** that are part of the rule set assigned to a patient are triggered on therapy data download.
- ✓ **Reminder sets** are a way to set a time-based notification for different patient sets for tasks such as follow-up, resupply or other.

Connected Trilogy Health Rules can be customized with patient-specific ventilator parameters to trigger notifications that proactively identify patients with potential increased need for intervention. In addition to usage rates, other Connected Trilogy Health Rules can be set for gross change and absolute threshold settings including:

- 1 Minute ventilation
- 2 Tidal volume
- 3 Respiratory rate
- 4 Ti/Ttot%
- 5 % Pt Triggered Breaths

Note: Multiple health rules can be combined into a rule set and applied to patients to enable disease specific protocols.

The screenshot shows the 'Care Orchestrator' interface with the 'Administration' tab selected. The 'Health Rule Sets' section is active, displaying a list of rule sets. The 'COPD Rule Set' is selected, showing its configuration. The 'When' condition is set to 'Minute Ventilation - Gross Change'. The 'Then' action is set to 'Add a task to follow up with the patient. With a Priority of Medium'. A graph shows the 'Baseline' (green line) and 'Evaluation' (blue line) periods, with a 'Change' (red line) indicating a significant increase in minute ventilation.



Efficiencies for everyone

Whether you're a equipment provider, doctor or clinician, Connected Trilogy with Care Orchestrator was designed to help you and your care team work together efficiently and reduce non-value added activities to help grow your business.

To integrate Connected Trilogy with Care Orchestrator into your workflow, use the following steps to make the transition easier:

- ✓ Let patients and care team know respiratory/ventilatory remote monitoring with Trilogy is now available
- ✓ Enhance your clinical protocols to include remote monitoring of key patient data
- ✓ Establish Health Rules and notifications with care team
- ✓ Establish communication process and protocols with care team



Connected Trilogy trial program

Philips Respironics is offering a 90-day trial period of Connected Trilogy service so you can evaluate the power and convenience of Connected Trilogy. This program allows you to gain experience with this solution before you purchase it. This is offered so you can better understand the full capabilities of Connected Trilogy and incorporate it into your clinical protocols.



Care across clinical care teams

The Connected Trilogy evaluation period includes:


- Up to five Connected Trilogy services per location-up to 10 locations
- 90-day period (three clinical cycles)
- No cost or commitment.

This offer is available to all customers who have not previously purchased Connected Trilogy

Flexible options to support your business needs

Full-service option	Part number
1 Year Connected Trilogy Service	1136869
3 Year Connected Trilogy Service	1136871

Purchase option	Part number
Qualcomm 2net Hub	1127528
Monthly Service	1134691

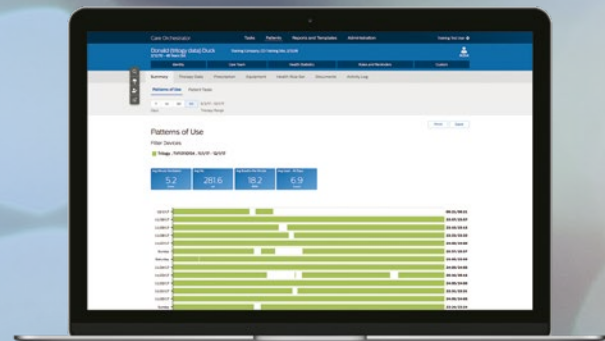



Learn more

Contact your Philips Respironics representative at **1 800 345 6443** or visit **philips.com/careorchestrator**.



© 2018 Koninklijke Philips N.V. All rights reserved.
Specifications are subject to change without notice.

www.philips.com/respironics

Caution: U.S. federal law restricts these devices to
sale by or on the order of a physician.

RRDPGH JM 10/22/18 MCI 4108537 PN 1140685
1010 Murry Ridge Lane, Murrysville, PA 15668
800 345 6443 • 724 387 4000