



ExamCard

Prodiva 1.5T* Prostate with fat-free mDIXON XD

ExamCard

Takahashi, Mamoru • Seirei Mikatahara General Hospital • Japan

Download ExamCard

Download

Please note that the ExamCard will be downloaded as a zip file.
You will need to unzip the ExamCard first, before you can use it on your MRI scanner.

Note: Change the "Save as Type" to "All Files" (default is "Text Document") to ensure that the ExamCard is saved in the correct format (.ExamCard).

ExamCard description

This Release 5.4 ExamCard for Ingenia Prodiva 1.5T* contains a comprehensive set of sequences for prostate imaging. Included are fat-free imaging sequences (mDIXON XD), providing multiple image types in 1 single scan, and multi b-value DWI (up to b2000).

ExamCard overview



<i>Images</i>	<i>Scan time</i>	<i>Voxels</i>
Axial T2w SPAIR	3:00 min.	0.8 x 0.9 x 4.0 mm
Axial T1w TSE mDIXON XD Water only and In Phase images	3:06 min.	0.9 x 1.0 x 4.0 mm
Axial DWI b800	2:57 min.	2.5 x 2.5 x 4.0 mm
Axial DWI b2000	2:45 min.	2.5 x 2.5 x 4.0 mm
Axial DWI ADC		2.5 x 2.5 x 4.0 mm
Coronal T2w SPAIR	3:00 min.	0.9 x 1.0 x 4.0 mm

* Not for sale in USA

[Contact](#) | [Philips](#) | [Terms of use](#)

© Koninklijke Philips N.V., 2017. All rights reserved.