



## Elition 3.0T Liver and MRCP - Amsterdam

### ExamCard

Nederveen, Aart, Ph.D. • Academic Medical Center • Netherlands

### ExamCard

Elition  
Release 5  
3D VANE, Body,  
Compressed SENSE,  
Liver, mDIXON XD,  
MRCP, MultiVane

### Download ExamCard Elition 3.0T Liver MRCP


The ExamCard will be downloaded as a zip file and needs to be unzipped before you can use it on your MRI scanner. Please use the Feedback button on the home page if you experience download problems.

### ExamCard description

This Release 5.4 ExamCard for Ingenia Elition X contains a broad set of sequences for liver imaging and MRCP on 3.0T. Included are:

- Coronal balanced TFE sequence
- Axial T2w TSE sequences with/without fatsat incorporating motion reduction technique (MultiVane XD)
- MRCP SSh and MRCP 3D with/without Compressed SENSE

### ExamCard overview

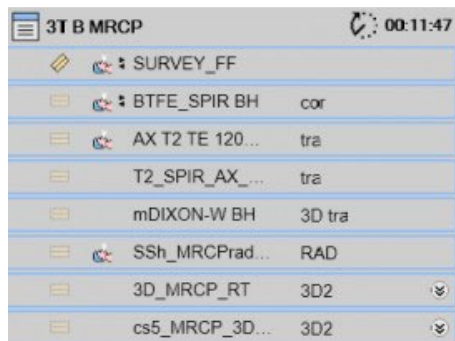

### ExamCard parameters

Sequences	Scan time	Voxel size
Coronal bTFE	0:40 min	1.5 x 1.8 x 7.0 mm
Axial T2w TSE	0:20 min	1.5 x 1.8 x 4.0 mm
Axial T2w SPIR MultiVane XD	2:36 min	1.7 x 1.7 x 4.0 mm
T1w mDIXON XD FFE Water only	0:10 min	1.7 x 1.7 x 1.7 mm
T1w mDIXON XD FFE In Phase	0:10 min	1.7 x 1.7 x 1.7 mm
T1w mDIXON XD FFE Out Phase	0:10 min	1.7 x 1.7 x 1.7 mm
MRCP SSh 15 radial slices	1:25 min	0.9 x 1.2 x 40.0 mm
MRCP 3D	4:06 min	1.0 x 1.2 x 0.9 mm
MRCP 3D Compressed SENSE	1:57 min	1.0 x 1.0 x 0.9 mm



**Elition 3.0T Liver and MRCP - Amsterdam**

This Release 5.4 ExamCard for Ingenia Elition X contains a broad set of sequences for liver imaging and MRCP on 3.0T.

**MRI • ExamCard**

By: [Nederveen, Aart](#),  
On: Jul 03, 2020

---

[Print](#)

---

[Contact](#) | [Philips](#) | [Terms of use](#)

© Koninklijke Philips N.V., 2017. All rights reserved.