

PHILIPS

DreamMapper

Getting started on DreamMapper

Take charge of your sleep therapy

Step 1 Download the DreamMapper app



Scan the QR code or search DreamMapper in your app store



Step 2 Register your DreamMapper account

a. Enter the required information and accept consents, before tapping **Register**.
Note: your email address will be your username.

b. Choose a password to meet the requirements. Review the optional consents and tap **Register**.

Create New Account

DreamMapper

John Smith

Australia

email@email.com

email@email.com

0000000000

IMPORTANT Ensure humidifier is empty before turning device over.

J222673378E65

Scan

I understand and accept the Terms and Conditions

I agree for my personal data to be processed in the DreamMapper App for creating my account and user profile and to provide a view of usage of my therapy device (s) to assist me with complying with my sleep therapy goals. Please review the attached Privacy Notice for more detailed information on what personal data we process and how we use data to provide you services. The Privacy Notice can provide you information on how we use data to provide DreamMapper services to you. I agree that my personal data can be processed to create my account and user profile.

Register Cancel



Register

Username

emailxxx@email.com

Password Requirements

- ✓ 8 - 20 characters
- ✓ Passwords cannot contain your email address or the word "password"

Passwords must have 3 of the following:

- ✓ Symbols
- ✓ Lower case
- ✓ Upper case
- ✓ Numbers

Acknowledge all consents

I would like to receive email communications, including advertisements and marketing, about Philips products, services, events and promotions.

It is ok to send me marketing advertising via app notifications – I can change this in my preferences anytime. What does this mean?

I agree to receive targeted marketing via app notifications that will be based on the sleep device I use. What does this mean?




I want to share my app analytics data to improve app functionality. What does this mean?

I agree that my de-identified data can be used to help improve the App, device and to develop new products and services. What does this mean?

Register Cancel

Step 3 Customise your account



a. Choose your mask type. Confirm by tapping **Set my mask**.

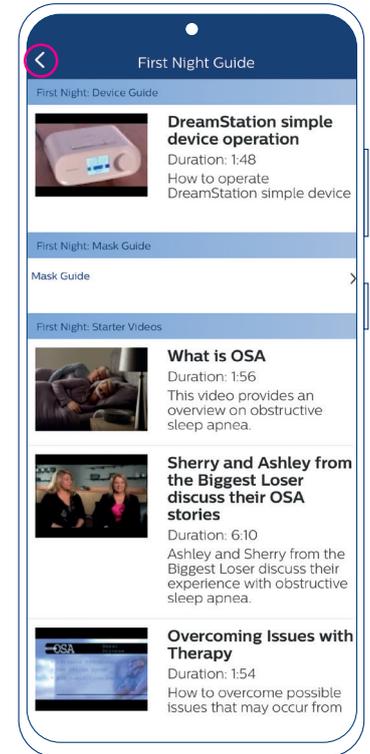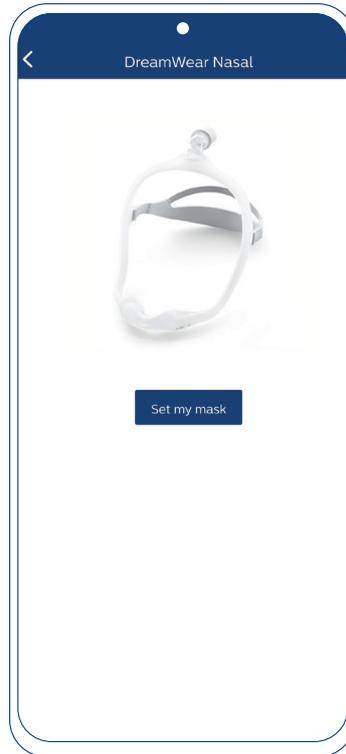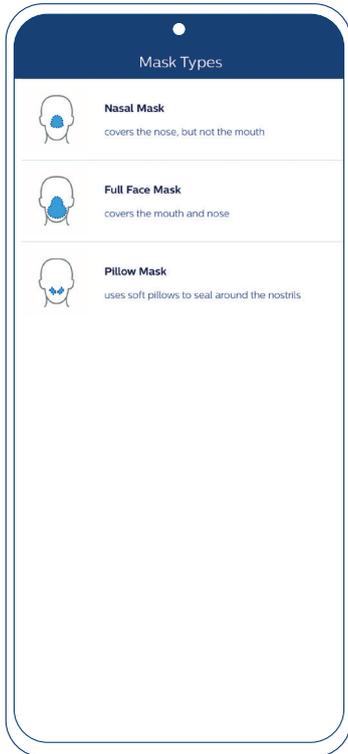

b. Review **First Night Guide** before clicking back arrow (on the top left).

Step 4 Bluetooth pairing for DreamStation

a. Tap **Yes** to allow your phone to use Bluetooth when pairing with your DreamStation.

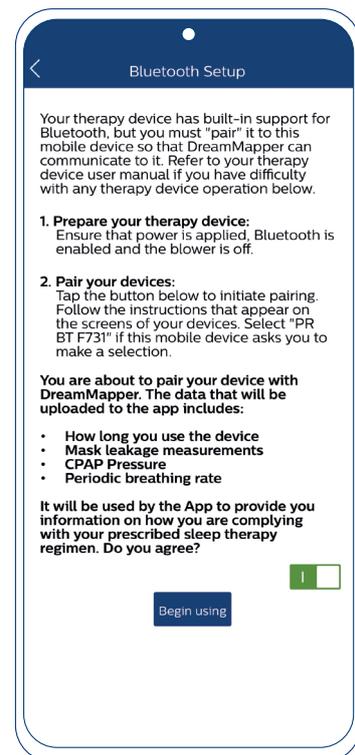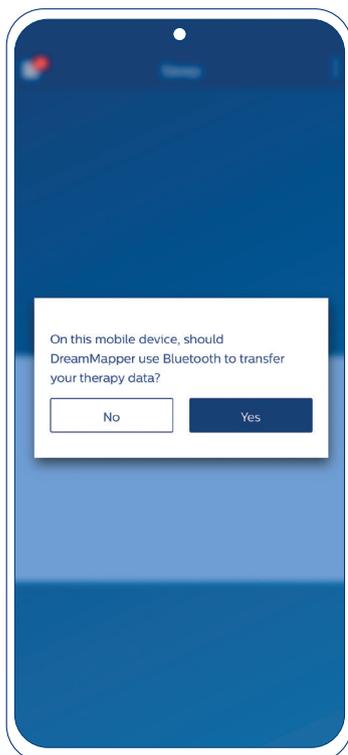


b. Read the instructions and tap the slider to green to consent. Then tap **Begin Using**.

c. On the DreamStation, turn the dial to **Yes** and push the dial to accept pairing.






d. Enter the code displayed on the DreamStation into the DreamMapper **Bluetooth Pairing Request** prompt on the app. Tap **Pair**.

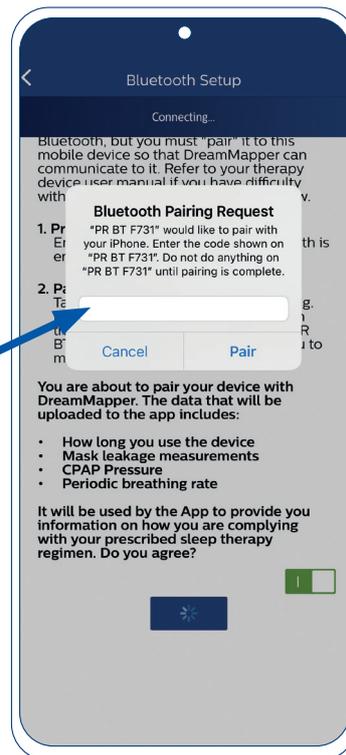

Step 5 Link DreamMapper to your sleep therapy provider

a. Tap **Yes** to consent to sharing your therapy progress with your sleep therapy provider. You will also need to supply them with the email address you used to register for DreamMapper.

b. Wait a moment for DreamMapper to connect to your DreamStation. DreamMapper should show the day of the week when set up is complete.

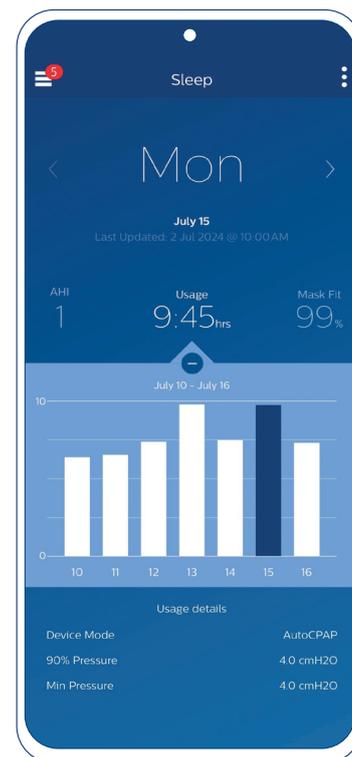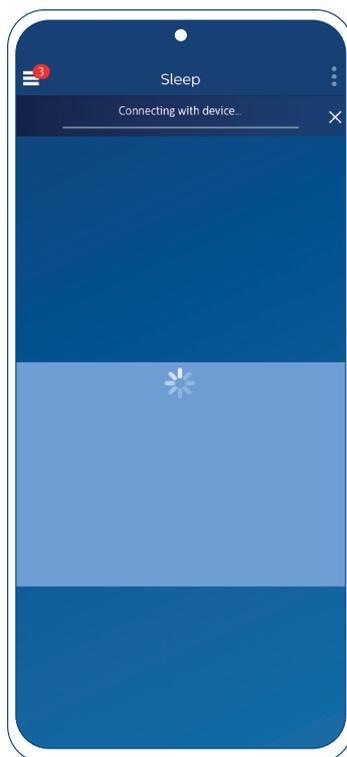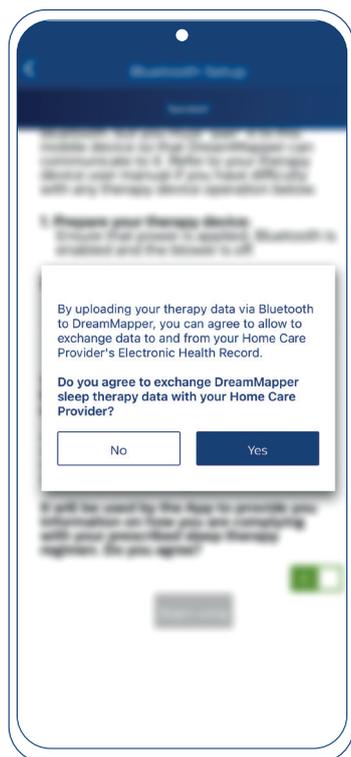

© 2024 Koninklijke Philips N.V. All rights reserved.
Specifications are subject to change without notice.
Trademarks are the property of Koninklijke Philips N.V.
or their respective owners.

00000497-07-01

Version: ANZ08/2024



Philips Sleep and Respiratory Care
65 Epping Road
North Ryde, NSW, 2113
Australia

www.philips.com.au/DreamMapper

Telephone: 1800 830 517 (toll-free)